

ComfiLife

FOR INSTANT HEAT

To begin the instantaneous heating process, simply “click” or “snap” the metal disc inside the gel. Within 5 seconds, the pack will heat up to a temperature of 130°F and will maintain this temperature for up to 30 minutes.

To reuse, return pack to its neutral state by soaking in boiling water until the salt crystals melt back into original liquid form and the pad is completely clear. When the pad is completely clear carefully take it out of the water and place it on a towel to cool down. The pad will be very hot after boiling. Flex metal disc to reactivate pack as needed.

Instant Heat Therapy Back Support (Maintains effectiveness with repeated use)

Just flex & use again and again!



Simply flex the
metal disc a
few times



Heat-activated
crystals form a gel



Providing heat
for up to 1/2 hours



For reuse: place
into boiling water

**To reuse, place in boiling
water until pack returns to
gel form.**

CAUTION / WARNING:

- Always consult a medical professional
- Read instructions carefully before each use
- Use only as directed. Do not apply hot pack directly to skin. Use pouch or wrap with towel before applying
- Handle carefully, hot pack will be very hot after boiling
- Do not apply to open wound or sensitive skin
- For external use only. Keep away from children, pets, elderly, or incapacitated people
- Should contents become exposed to the skin or eyes, wash with water immediately and consult a physician
- Discard if leaking or damaged
- Do not use while sleeping

Visit: www.comfilife.com For warranty registration and additional information.