

# THANK YOU!

We appreciate your trust in our products, we are ecstatic that you chose ComfiLife Knee Brace over many other possibilities out there. You will soon realize that you have made the best decision!

Now sit back and relax, we've got your back!

## Not Satisfied?

Contact us

### Satisfaction Guaranteed

At ComfiLife we take pride in the quality of our products and our customer service. Your purchase comes with a 100% satisfaction and replacement guarantee.

If you are not 100% satisfied for any reason or if there are any issues with your knee brace please contact us and we will send you a FREE REPLACEMENT.

For any questions please contact us and our lightning speed customer service will take care of you.

### Please Read Before Use

- Your knee brace fits left or right knee. It is normal to feel some tightness around the knee during the first few uses. The fit should be snug and not too tight, it should not restrain you from doing your daily activities.
- Open patella design with padded ring provides support for the knee cap while adjustable straps provide custom compression support around the knee and prevent slippage during sport activities.
- The knee brace straight from the package may have a slight odor. This is normal, air it out for a few hours and wash before use.
- Always consult with your physician before using a knee brace.

### Other Best Sellers From ComfiLife

ComfiLife Ankle Support Wrap



ComfiLife Knee Sleeve



ComfiLife Patella Brace



ComfiLife Hot & Cold Pack



ComfiLife Knee Ice Pack



# How to use?



Place the brace on your knee so that your knee cap is centrally aligned with the patella opening.



Wrap the middle strap behind the knee and fasten it.



Fasten the top strap above your knee cap.



Fasten the bottom strap below your knee cap.



Adjust the straps to achieve your desired compression.



The fit should be snug but not too tight.

## Care Instructions

• Hand wash in cold water. Wash gently, do not scrub. Do not use fabric softener, bleach or any harsh chemicals. Air dry and make sure your brace is completely dry before putting it on.

**WARNING: DO NOT PUT YOUR BRACE IN THE DRYER OR ON A HEATER. DO NOT USE HARSH DETERGENTS OR BLEACH PRODUCTS.**



Do Not Dry Clean



Do Not Bleach



Hand Wash Cold



Air Dry



Do Not Iron

Visit: [www.ComfiLife.com](http://www.ComfiLife.com)

**within 14 days to register your product for full warranty**