


THANK YOU!

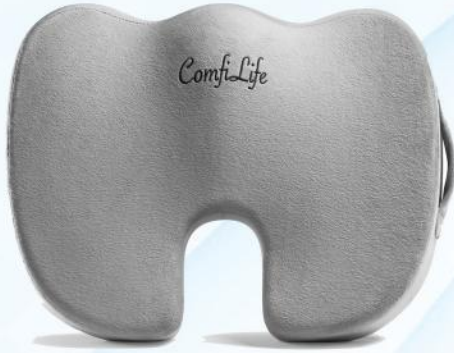

We appreciate your trust in our products, we are ecstatic that you chose ComfiLife Seat Cushion over many other possibilities out there. You will soon realize that you have made the best decision!

Now sit back and relax, we've got your back!

Not Satisfied?

Contact us

Satisfaction Guaranteed

At ComfiLife we take pride in the quality of our products and our customer service. Your purchase comes with a 100% satisfaction and replacement guarantee.

If you are not 100% satisfied for any reason or if there are any issues with your cushion please contact us and we will send you a FREE REPLACEMENT.

For any questions please contact us and our lightning speed customer service will take care of you.

Please Read Before Use

- Memory Foam is heat responsive, in order to get the best performance, keep your cushion at room temperature at all times.
- Your cushion will conform to your body within a few days.
- Freshly manufactured memory foam might have a slight odor. Air out your cushion in a well ventilated area for at least 24 hours. If possible let it sit outside for a few hours on a dry sunny day.

Other Best Sellers From ComfiLife

ComfiLife Lumbar Pillow



ComfiLife Knee Pillow



ComfiLife Anti Fatigue Mat



ComfiLife Pillow Protectors



ComfiLife Gel Seat Cushion





How to use?

- Place your cushion with the **cut-out towards the back of your chair**. When you sit on your cushion align your tailbone with the cut-out.
- Your cushion might raise you slightly, adjust your seat to a comfortable height or use foot stool to make sure your lower body is in a comfortable position.
- Always consult with your physician for the best suitable product for your specific condition.

Care Instructions

- The cover of your ComfiLife Cushion is removable and washable. However the memory foam is not washable. You must remove and wash the cover only.
- Hand wash in cold water. Do not use fabric softener, bleach or any harsh chemicals. Line dry and make sure the cover is fully dry before use.



Do Not Dry Clean



Do Not Bleach



Hand Wash Cold



Line Dry



Do Not Iron

Your Opinion Matters

At ComfiLife we take pride in the quality of our products and our customer service. As a small business we would love to hear your honest feedback and we know other shoppers would too. So when you have a few minutes can you let us know what you think?

Leaving a Review is easy:

- 1) Log into your Amazon Account
- 2) Go to Your Orders
- 3) Write Product Review

If you have any questions please contact us and our lightning speed customer service team will take care of you.

Visit: www.ComfiLife.com

within 14 days to register your product for full warranty