

# THANK YOU!



We appreciate your trust in our products, we are ecstatic that you chose ComfiLife Wrist Support Wrap over many other possibilities out there. You will soon realize that you have made the best decision!

Now sit back and relax, we've got your back!

## Not Satisfied?

Contact us

### Satisfaction Guaranteed

At ComfiLife we take pride in the quality of our products and our customer service. Your purchase comes with a 100% satisfaction and replacement guarantee.

If you are not 100% satisfied for any reason or if there are any issues with your wrist wrap please contact us and we will send you a FREE REPLACEMENT.

For any questions please contact us and our lightning speed customer service will take care of you.



### Please Read Before Use

- Your wrist wrap fits left or right wrist. It provides a custom compression, the fit should be snug and not too tight.
- Your wrist wrap is made of a breathable fabric for ultimate comfort however skin irritation may occur between the thumb and the index finger for the first few uses. To avoid this pat an ample amount of talcum powder or cornstarch to the affected area before use.
- Constant use may stretch the material of your wrist wrap, washing it would restore the firmness of the elastic fabric.
- Always consult with your physician before using a wrist support brace.

### Other Best Sellers From ComfiLife

ComfiLife Wrist Strap



ComfiLife Ankle Support Wrap



ComfiLife Knee Sleeve



ComfiLife Hot & Cold Pack



ComfiLife Knee Ice Pack



# How to use?



**1**  
Put your thumb through the loop. Pull the strap down and wrap around your wrist.



**2**  
Continue wrapping around your wrist and pull it for a snug but not too tight fit.



**3**  
While wrapping the strap, adjust the compression to your needs.



**4**  
Once the compression is adjusted fasten the wrap on itself.

## Care Instructions

• Hand wash in cold water. Wash gently, do not scrub. Do not use fabric softener, bleach or any harsh chemicals. Air dry and make sure your wrap is completely dry before putting it on.

**WARNING: DO NOT PUT YOUR WRAP IN THE DRYER OR ON A HEATER. DO NOT USE HARSH DETERGENTS OR BLEACH PRODUCTS.**



Do Not Dry Clean



Do Not Bleach



Hand Wash Cold



Air Dry



Do Not Iron

Visit: [www.ComfiLife.com](http://www.ComfiLife.com)

**within 14 days to register your product for full warranty**